

C-FORM

Conforming Cast Tape

The closest you will come to Plaster in a Synthetic Cast Tape

EASY TO CAST

VERY CONFORMING

C-FORM Features:

- Conforms nicely to complex shapes
- Allows for the practitioner to produce a more conforming cast.
- No plaster mess
- Ample time to work with, before it hardens
- No need for special glove. A regular casting glove is fine to use
- Resin does not stick to the glove like typical fiber glass casting tape
- Rigid cast once finished
- Cost effective (comparable conforming casting tapes)

Product Description BOX = 10 Rolls	Product #
C-Form Casting Tape - White 2" x 4yd	(4013-2" Box)
C-Form Casting Tape - White 3" x 4yd	(4013-3" Box)
C-Form Casting Tape - White 4" x 4yd	(4013-4" Box)
C-Form Casting Tape - White 5" x 4yd	(4013-5" Box)



CASTING INSTRUCTIONS USING C-FORM

1. Take the roll out of the package and immediately dunk into the water.
2. Make sure the water penetrates it thoroughly. Squeeze the roll hard while in the water to help penetrate and set off the resin within the water.
3. Take the roll out of the water and squeeze lightly the excess water. Key is to keep the roll wet.
4. When removing, If using a cutting blade, lay the blade to the side at a 40+ degree angle and cut.

Tips:

- Tip #1 - The first layer of C-FORM casting tape should be rolled gently. Then gently stretch second layer to release resin from casting tape. The resin is built into the fiber.
- Tip #2 - When you feel the heat from the resin setting off, this is when you want to shape the cast to fit / conform to the patient's limb.
- Tip #3 - USE **WARM** water for faster activation.

Distributed by:



Atlas International
11389 Trade Center Drive
Rancho Cordova, CA 95742

www.atlasortho.com
(800) 545-6287